



and

MedE-Kast[®]
TOTAL CONTACT CAST

MedE-Kast[®] Ultra
TOTAL CONTACT CAST



DERMASCIENCES
A TISSUE REGENERATION COMPANY
214 Carnegie Center, Suite 300
Princeton, NJ 08540
USA: 800-825-4325
Canada: 800-387-5302
www.dermasciences.com
www.TCCEZ.com
www.TCCPatient.com

Customer Service
800-825-4325

Evenup[®] is a registered trademark of Evenup, LLC.
TCC-EZ[®], MedE-Kast[®], and MedE-Kast Ultra[®] are registered trademarks of Derma Sciences, Inc. ©2015 Derma Sciences, Inc.



Accompanied with...



Evenup[®] is:

- To be used on opposite foot from TCC-EZ[®] or MedE-Kast[®].
- Enables easier ambulation for some casted patients.

Safety Precautions:

- Be careful! Your balance is likely compromised due to your injury or surgery.
- Walk slowly with the Evenup[®] to prevent falls.
- Do not walk on soft or uneven ground such as sand, gravel, or grass while wearing your Evenup[®].
- While wearing TCC-EZ[®], with or without Evenup[®], a four prong cane or walker might be recommended.
- Always wear your Evenup[®] securely fastened to a shoe.
- Remove the Evenup[®] from your shoe before operating a motor vehicle.
- Recommended for use with laced oxford, orthopedic shoes or sneaker.

Instructions:

Determine the thickness needed. The single bottom sole is 1.3 cm thick. Adding the top sole the Evenup[®] will be 2.1 cm total in height. To join the two layers using the Velcro, simply line up the two strips on each sole and press firmly together. Keep in mind, different shoes have different sole thicknesses, one shoe may require both layers while a different shoe may only require one layer.




1. Put on your Evenup[®] while sitting down. Place your good foot over your opposite knee. Place the front portion of the Evenup[®] strap over the toes of your shoe.



2. Pull the elastic straps toward the heel of the shoe and adjust the side straps over the sole of the shoe. Pull up the heel tab to securely fasten to the sole of the shoe.



3. Make sure the Evenup[®] name is facing out.

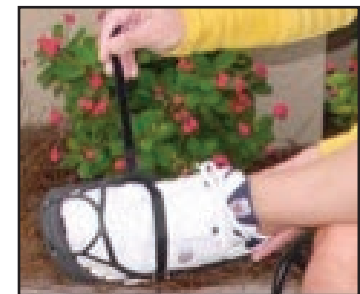

4. Utilize the velcro strap provided to securely fasten the Evenup[®] to your shoe. Place the strap through the loop and around the thicker band.